McGRATH™

MAC

INSTRUCTIONS FOR USE

en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
da	Brugsanvisning
cs	Návod k použití
sk	Návod na použitie
ro	Instrucțiuni de utilizare
et	Kasutusjuhised
hr	Upute za uporabu
sr	Uputstvo za upotrebu
id	Petunjuk Penggunaan

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Description

The McGRATH™ MAC video laryngoscope (device) is a tool used to aid the intubation of the trachea. As a rigid laryngoscope it holds and shapes the anatomy allowing a clear view of the larynx and entrance to the trachea.

The device incorporates a light source (LED) and miniature camera (camera) to view the larynx during the procedure of laryngoscopy. The image is displayed on an LCD screen (screen) contained within a monitor mounted to the handle of the device.

A McGRATH™ MAC 3.6V battery (battery unit) mounted within the handle powers the screen, camera, and LED. The McGRATH™ disposable laryngoscope blade (blade) covers the camera and LED assembly (CameraStick™) to prevent direct patient contact. Blades are supplied sterile and are single use.

The use environment may include: hospitals, hospital-type facilities, intrahospital transport, mobile emergency medical applications including both ground and air transport. Transport environments include intra-hospital transport and both ground and air emergency transport: road ambulances and fixed-wing aircraft and helicopters.

Accessories

340-000-000	McGRATH™ 3.6V Battery
350-072-000	McGRATH™ MAC 1 disposable laryngoscope blades
350-017-000	McGRATH™ MAC 2 disposable laryngoscope blades
350-005-000	McGRATH™ MAC 3 disposable laryngoscope blades
350-013-000	McGRATH™ MAC 4 disposable laryngoscope blades
X3-003-000	McGRATH™ X blade X3 disposable laryngoscope blades

All blades described above are regulated under  0123

For ordering please contact your local representative.

Intended Use

The McGRATH™ MAC video laryngoscope and McGRATH™ disposable laryngoscope blades are intended to be used by trained and licensed individuals to gain a view of the vocal cords during medical procedures, typically laryngoscopy.

Indications

The visualization of the oropharynx and larynx for general laryngoscopy and/or to aid in the securement of airway.

Contraindications

None known

Warnings

- **Blades cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and are therefore intended for single use. Attempts to clean or sterilize the blades may result in bio-incompatibility, infection, or product failure risks to the patient.**
- **The device must be used only with McGRATH™ disposable laryngoscope blades.**
- **Check that the blade is correctly fitted to the CameraStick™ before use.**
- **If the blade clip does not securely engage onto the CameraStick™ do not attempt to use the blade.**
- **If there is any evidence that the blade packaging has been opened, or damaged, do not use the blade.**
- **Only personnel trained and licensed to perform intubation with a laryngoscope may use this device.**
- **Do not use the blades with any other type of laryngoscope. Blades are designed to be used with the McGRATH™ MAC video laryngoscope only.**
- **Do not open any sealed part of the device. Doing so will impair the performance of the device and void the warranty, and may impact patient safety.**
- **Do not use if any components become loose, physically damaged, or fit poorly.**
- **Ensure the device is completely dry before use.**
- **Do not autoclave the device.**

- **Do not reprocess this device in an ultrasonic cleaner.**
- **Always check that the device is working correctly before each patient use.**
- **Remove the battery prior to cleaning and/or performing disinfection.**
- **Do not modify the device.**
- **Portable radio frequency communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to the device.**
- **McGRATH™ MAC 3.6V battery**
 - **Fire, explosion, and severe burn hazard.**
 - **Do not recharge, disassemble, heat above 212°F (100°C), incinerate, or expose device to water.**
 - **Do not crush.**
- **Do not use in an MR environment. Attempting to use the battery in an MR environment could cause burns or adversely affect an MR image.**
- **Ensure that local handling procedures are followed at all times while handling disposable devices.**
- **The light emitting portion of the video laryngoscope may exceed 41°C as part of normal operation. When used as indicated, continuous contact with this area is unlikely.**

Cautions

- **Do not use any other type of laryngoscope blade with this medical device.**
- **Do not use in the presence of flammable mixtures.**
- **Do not use any other type of battery with this device.**
- **Federal (USA) law restricts this device to sale by or on the order of a physician.**

Adverse Events

The following adverse events have been reported to be associated with the use of laryngoscopes during laryngoscopy and/or intubation listed in order of frequency and severity: mucosal injury, delay of treatment, hypoxia, esophageal or mal-positioning of tracheal tube, change in vital signs, dental injury, intubation related aspiration, bleeding, infection, discomfort, and foreign body.

If a serious incident related to the device occurs, immediately report the incident to Aircraft Medical and the applicable competent authority.

Specifications

Video Laryngoscope Device	
Size:	180mm x 68mm x 110mm
Weight:	0.180kg
Internally Powered:	Proprietary McGRATH™ 3.6 V Lithium Battery *Giving 250 minutes of use typically
Protection:	IP38 Please pay attention to the effects of lint, dust, light (including sunlight), etc.
Light source:	LED
Display:	2.5" LCD colour display
Camera:	CMOS
RFID:	Transmitting frequency: 13.56MHz Output power: 70mW
CISPR	CISPR 11 Group 1 Class B
Useful life:	4 years or 2000 decontamination cycles.
*Where the ambient temperature is 20°C, battery life may vary where the ambient temperature is higher or lower. Do not store batteries or devices with batteries at more than 40°C (104°F). Doing so will shorten the life of the battery.	
Disposable Laryngoscope Blade	
Material:	The blades are disposable, composed of medical grade optical polymer which have not been made with natural rubber latex and not made with DEHP. Blades are packaged individually in pouches that have not been made with natural rubber latex.
Applied Part Type:	BF

Note: Dispose of device, battery, and blade according to local regulations.

Operating Conditions

Device temperature	10°C (50°F) to 40°C (104°F)
Atmospheric pressure	620 hPa to 1060 hPa
Relative humidity	10% to 95% (non-condensing)

Storage and Transportation

Device and Battery	
Device temperature	-10°C (14°F) to 40°C (104°F)
Atmospheric pressure	620 hPa to 1060 hPa
Relative humidity	10% to 95% (non-condensing)
Blade	
Device temperature	-10°C (14°F) to 70°C (158°F)

Unpacking, Inspection

Unpack device and check for any visible signs of damage. In the event of signs of damage, do not use the device and inform your local representative. Remove the protective sheet from the monitor screen and camera cover from the distal end of the CameraStick™. The device and battery unit are supplied separately within the same container in a non-sterile state. Decontaminate the device and battery before first patient use, as instructed in the cleaning and disinfection section.

Operation

The device turns on and off by a single push of the power button. Always check that the device is working properly and that there is sufficient remaining power in the battery prior to starting a clinical procedure.

Battery

The life of the battery unit is displayed on the bottom right corner of the screen. Refer to quick start guide included with the device for graphic example.

When the battery counter reaches 5 minutes, the battery icon will flash. Replace the battery unit when this happens. If the counter is allowed to run down to zero, the battery icon will continue to flash. Pull on the tab to remove the battery.

Power Saving

To save power, the device will automatically switch off when motionless for a period of time. After 15 seconds of inactivity, the device screen will dim in preparation for switch off. The device reactivates when it detects movement. However, if the period of inactivity continues for a total of 3 minutes, the device will switch off. When this occurs, the device can be switched back on by a single push of the power button.

Disposable Laryngoscope Blade Fitting

To fit the blade, open the peel pouch and slide the blade over the CameraStick™ portion of the device. The blade is correctly fitted when the blade clip is firmly latched to the CameraStick™. Remove the blade from the peel pouch just prior to intubation. To remove the blade after intubation, lift the clip, pull the blade off and dispose of according to local regulations.

Using the Laryngoscope

Multiple disposable blade types are available for use with the McGRATH™ MAC Video laryngoscopes. The McGRATH™ MAC range of disposable blades are for use with routine and difficult airways. The McGRATH™ X blade™ range are for use with difficult and extreme airways. These blades require different techniques which are explained below.

McGRATH™ MAC blade (for graphical representation, refer to quick start guide included within the packaging)

1. If possible, position the patient in the optimal position for direct laryngoscopy.
2. Look into the mouth; insert the blade into the right side of the mouth.
3. Move the device to a central position while sweeping the tongue to the left.
4. Advance the tip of the McGRATH™ MAC blade into the vallecula.
5. Visualize the epiglottis on the screen. Lift the anatomy forwards and upwards to expose a direct and indirect view of the glottis. When the device is in the optimal position the glottis should be viewed in the central upper section of the screen.

6. Advance the E.T. tube gently and atraumatically through the vocal cords. Tube placement can be performed either by looking directly in the mouth, indirectly on the screen or a combination of both*.
7. Indirectly visualize the E.T. tube placement through the vocal cords. In optimal tube placement technique, the E.T. tube will enter from the right side of the display.
8. The screen view can be used to confirm the correct insertion depth of the E.T. tube.

*If a direct pathway for the E.T. tube was not created by sweeping the tongue or aligning the airway axes, a stylet or a bougie may be used.

McGRATH™ X blade™ (for graphical representation, refer to quick start guide included within the packaging)

1. Load the E.T. tube onto a stylet and form to the curvature of the X blade™. Clinical experience has shown that intubation with a stylet is required to facilitate optimal tube placement. Use of a bougie is not advised. Where possible, position the patient's head for optimal access.
2. Using a mid-line approach, roll the blade into the mouth. Ensuring the anterior side of the blade maintains contact with the tongue, advance the blade until the epiglottis is seen on the top of the screen.
3. Place the tip of the X blade™ into the vallecula. Using minimal force, rock the device back towards the user to lift the epiglottis and obtain an indirect view of the glottis. When the device is in the optimal position, the glottis will be viewed in the central upper section of the screen*.
4. The DepthGuide™ numeric markings on the posterior side of the blade may be used as an indication of the depth of blade insertion**.
5. Insert the E.T. tube at the right side corner of the mouth. Advance in a rolling movement following the curvature of the blade, ensuring it maintains contact with the section of the blade labelled E.T. CONTACT ZONE™.
6. When using optimal technique, the E.T. tube should enter the screen on the right side; advance the tube until the tip is in front of the vocal cords.
7. Holding the stylet secure, slide the tube off the stylet and through the cords, ensuring the stylet does not pass through the cords. Once the E.T. tube has passed the cords, remove the stylet completely.

8. The screen view can be used to confirm the correct insertion depth of the E.T. tube.

*It is important not to advance the blade too deeply in order to maintain maximum space to facilitate the E.T. tube placement.

**Reference to these numbers can be useful during training to avoid inserting the blade too far.

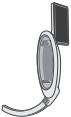
Cleaning and Disinfection

After unpacking, decontaminate the device and battery before use. The device and battery should be cleaned and disinfected separately after each patient use. Cleaning and disinfection should be carried out as per this instructions for use document and in accordance with local regulations. Follow hospital policy and protocol when handling and cleaning soiled items.

The instructions provided have been validated by Aircraft Medical as being capable of preparing the device for re-use.

The following methods have been approved:

Note: Blades are not intended to be cleaned or disinfected.

Product	Cleaning	Intermediate Level Disinfection
McGRATH™ MAC with McGRATH™ 3.6V Battery removed 	Quaternary ammonia + isopropyl alcohol (IPA) (e.g. Super sani-cloth or equivalent)	1. Quaternary ammonia + isopropyl alcohol (e.g. Super sani-cloth or equivalent)
Product	Cleaning	Low-Level Disinfection
McGRATH™ 3.6V Battery 	70% IPA wipe	70% IPA wipe

Dispose of wipes in accordance with local regulations and hospital policy.

Device Cleaning

Use the following process to clean the device:

1. Ensure the battery is removed prior to cleaning.
2. Use a quaternary ammonium and isopropyl alcohol wipe (e.g., Sani-Cloth or equivalent).
3. Clean the device, systematically working from Step 3a through Step 3g (from top to bottom of the device). - Ensure the surfaces remain thoroughly wet by continuously wiping for 1 minute.
 - a. Monitor: Ensure the disinfectant wipe gets into the various grooves around the screen.
 - b. Monitor Hinge: Rotate the monitor to the upright position. Feed the disinfectant wipe into the space between the monitor and handle to ensure effective penetration. Rotate the monitor to its opposite position and repeat.
 - c. Battery Bay: Ensure all surfaces of the Battery Bay (with battery removed) are thoroughly treated with the disinfectant wipe. Ensure that the joint between the battery module and the handle is thoroughly treated with the disinfectant wipe.
 - d. Handle: Ensure all surfaces of the handle are thoroughly treated with the disinfectant wipe.
 - e. Clip and Heel Area: Take particular care in ensuring that the small clip is clean, in particular the internal corner between the clip and the surrounding plastic body.
 - f. Camera Stick™: Ensure all surfaces of the camera stick are thoroughly treated with the disinfectant wipe.
 - g. Camera Lens: Clean the camera lens with the disinfectant wipe. Ensure that the interface between the Camera Stick™ and camera lens is clean.
4. Visually inspect the device for visible soil. If not visually clean, repeat cleaning from step 3 using a new disinfectant wipe.

Device Intermediate-Level Disinfection

Use the following process to disinfect the device

1. Ensure the battery is removed prior to disinfection.
2. Use quaternary ammonia + isopropyl alcohol wipe (e.g. Super sani-cloth or equivalent).

3. Follow the same process as per the device cleaning section above to intermediate-level disinfect the device. Using a new wipe, disinfect the device, systematically working from Step 3a through Step 3g of the device cleaning section above (from top to bottom of the device). Ensure that the wipe comes in contact with all surfaces of the device for the contact time recommended per the wipe manufacturer's instructions. After reprocessing, wipe the battery contacts.

McGRATH™ 3.6V Battery Cleaning

Clean with 70% IPA wipe

Use the following process to clean the battery:

Using a 70% IPA wipe, clean the battery, systematically working through the following six steps (from top to bottom of the battery cover). Ensure that the 70% IPA wipe comes in contact with all surfaces of the battery. Ensure a minimum contact time of 1 minute on all surfaces.

- Carbon Pill: Ensure the 70% IPA wipe gets into the two button slots at the back of the battery push button.
- Power Contact: Ensure the 70% IPA wipe gets into the space around and under the two metal battery contacts.
- Clips: Run the 70% IPA wipe along and into the gaps around the battery retaining clips
- Lower Battery Slot: Ensure the 70% IPA wipe gets into the lower slot at the base of the battery
- Battery: Wipe all surfaces of the battery cover with the 70% IPA wipe
- Tab: Run the 70% IPA wipe along all surfaces of the tab.

Repeat cleaning where visible soiling is still present using a new 70% IPA wipe. Ensure that the 70% IPA wipe comes in contact with all surfaces of the device even if no visible soiling is present.

McGRATH™ 3.6V Battery Low-Level Disinfection

Disinfect with 70% IPA wipe

Follow the same process as per the battery cleaning section above to low-level disinfect the battery.

Using a new 70% IPA wipe, disinfect the device, systematically working through the six steps of the battery cleaning section instructions (from top to bottom of the battery).

Decontaminated batteries should be packaged immediately upon completion of the 2 minute drying phase and should be stored in accordance with local guidelines.

Storage

After cleaning and decontamination, the device and battery should be air dried or wiped dry with a lint free cloth. The device has to be completely dry before use.

Decontaminated devices should be packed immediately after drying and stored in accordance with local guidelines. The device cannot be stored in its original packaging for reuse.

Troubleshooting

System Messages

For system messages examples see quick start guide included within the packaging.

- a. Normal screen (250 minutes of battery run time available on battery) Figure 2
- b. 5 minutes of battery run time remaining remaining on the battery (it is recommended to replace the battery) Figure 3
- c. Unknown minutes of battery runtime available (this is used after the '0' minutes or if there is an internal error after startup) Figure 5
- d. Dimmed, device is not sensing any motion. If it remains in this state, the device will turn off. (after 3 minutes of no motion)
- e. System error, this screen shows if there is a startup error including if there are 0 (zero) minutes available on the battery Figure 4

No Image Shown On Screen or visible damage to the power button

- Replace battery unit.

Poor Picture Quality

If the image displayed on the screen is blurred or fuzzy:

- Remove the blade and check that the image is clear. If necessary wipe the camera at the end of the CameraStick™ with a lint free wipe.
- Replace the blade.

If neither of the above achieves positive results, contact local representative for diagnostics and repair. There are no serviceable parts inside the device.

During use with MAC blade, if the camera system loses function and the LED remains functioning, the device can still be used as a direct view Laryngoscope. During use with the MAC blade, a failure of the LED requires the removal of the Video laryngoscope and an alternative device to be used. During use with the X-Blade™, any failure of the camera system or LED requires the removal of the Video laryngoscope and an alternative device to be used.

Unexpected Video Interruption

- Try to turn the device back on.
- Replace the battery unit.

Warranty

The device is supplied with a manufacturer's warranty.

- Only products supplied by an approved Aircraft Medical agent or distributor are covered by the manufacturer's warranty.
- To be covered by the warranty the product must be maintained in accordance with the procedures in this document.
- Before returning the device under a warranty claim, disinfect the device per the instructions in this document.

For full terms of warranty, please contact your Aircraft Medical agent or distributor.

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Aircraft Medical Ltd., 9-10 Saint Andrew Square,